

Vaporizing for singers and speakers.

It is proven that Vaporizing with a salt water solution works in favor of the recovery after a heavy duty or throat issues for the vocal chords.

By inhaling the salty moist air the salty liquid will be on the vocal chords which provides a relaxation around the vocal chord and helps the stressed muscles in the larynx to relax.

Test

In a test 20 opera singers were asked to sing full power in a very dry room. The singers were having serious problems with their voice due to the dry air. Some became hoarse and some just felt heaviness when trying to sing.

Recovery:

After 2 hours of only vocal rest nobody recovered and most of the singers had serious issues trying to sing. But after 2 hours of rest in combination with vaporizing the singers gave us feedback to be almost or completely recovered.

When do you Vaporize?

It is ideal for the air-conditioned surroundings that we sometimes have to perform in. Airplanes can ruin the voice, and halls where they use smoke (Oil based) on stage. During heavy-duty touring or singing heavy-duty repertoire. In all these cases vaporizing will help to have a healthy working instrument.

The product:

The Vaporizer we sell is very small (4,7 Inch) and easily to hold in your hand. It makes no sound so nobody will be bothered when using it. It fits in any bag and is easy to travel with. Be sure you use the NaCl (0,9%) and put it in the big compartment in the top front.

It is delivered with 2 AA batteries so you can use it anywhere you want. It has different mouthpieces like a mask or a tube. It is up to you which one you like the best

The price is €109,- including 6% Tax (Around \$125,-)

Including 30 ampoules €118,- inclusive 6% tax (Around \$133,-)

An invoice will be sent to you through e-mail. After paying we will ship the vaporizer.

Shipping will have an added amount depending on the country we have to send it to. We will send you an email with the total cost before we send you the invoice

How long?

It is not possible to vaporize too long because it has no downside. The NaCl is compatible with the salt percentage of a human body. But a minimum of 10 minutes is recommendable. Try to do it 2 or 3 times a day if you are in a situation where your voice is under working stress.

Maintenance of the vaporizer.

The Batteries included are so called 'demo's' and will wear out fast. We recommend you to put high quality Batteries in as soon as possible. After vaporizing you ALWAYS empty the reservoir and put clean water in. Then start vaporizing (No inhalation) so the salt water will be flushed out of the very tiny holes in the MESH. (otherwise it might crystallize) After about 15 times using the Vaporizer it is wise to clean it once with white vinegar. Just put it in top and put the vaporizer ON. Then after 5 minutes of vaporizing (No inhaling!!) you empty the reservoir and put the clean water in top to flush the Vinegar out.

You can take the head of the Vaporizer off by pushing the button on the back of the top. It is possible to lay the top completely in the white vinegar. Off course it needs proper drying afterwards.

Ordering

You can order the vaporizer by using this form: <http://www.vocalcenter.nl/zang-informatie/stemonderhoud/vernevelen>

You can also send us an e-mail at info@vocalcenter.nl

We hope you will enjoy the vaporizer.

Team Vocal Center.

Stichting Vocal Center

Moerasbos 24
3411 WD Lopik

KVK: 56332866
BTW: NL852077403B01
IBAN: NL88RABO0132269805
BIC: RABONL2U
Website: www.vocalcenter.nl

Email: info@vocalcenter.nl